


# Sani-Cloth® AF: Instructions for Use

## Two in one cleaning & disinfectant alcohol-free wipes

For non-invasive medical devices and non-porous hard surfaces. It can be used to pre-clean or decontaminate medical devices prior to sterilisation. Where surfaces are soiled, pre-cleaning is required. For professional use only.



### Preparation

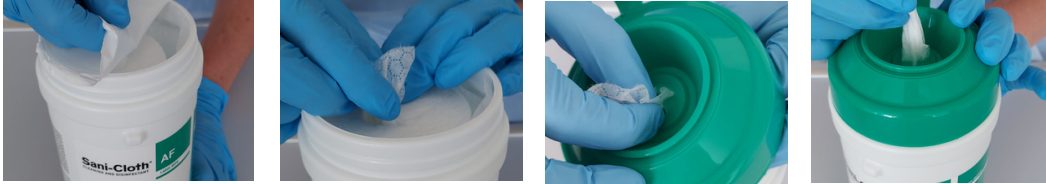

### Dispense




- + Dispense the wipe at a 45° angle, pulling away from you.
- + Close the lid when not in use.

### Clean



- + If the surface is visibly soiled, clean the surface following an S-shaped motion from top to bottom.
- + Use additional wipes as needed.
- + Use a fresh wipe for each new surface.

### Disinfect

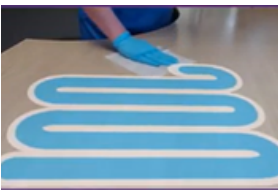

- + Using a fresh wipe, disinfect the surface in an S-shaped motion from top to bottom, avoiding cross-contamination.
- + Use a fresh wipe for each new surface.

### Contact Time



- + Ensure the surface remains visibly wet for 2 minutes for complete disinfection.
- + Use an additional wipe if necessary.
- + Allow to air dry.

### Disposal



- + Dispose of wipe after a single use.
- + Packaging is recyclable.



# Sani-Cloth® AF: Instructions for Use

## Two in one cleaning & disinfectant alcohol-free wipes

### Caution & Other information:

- + Store in a well-ventilated place. Keep cool
- + Avoid release to the environment. Harmful to aquatic life with long-lasting effects
- + Refer to the expiry date on the packaging.
- + Approx.1 wipe per 1m<sup>2</sup>.
- + Dispose of contents and container in accordance with local regulations

### Contra-indications:

- + Do not use for terminal disinfection of invasive medical devices.
- + Any serious incident that occurs in relation to the device should be reported to the ANZ Distributor
- + Compatibility with medical devices should be checked before use



### Product Efficacy

Sani-Cloth® AF	Test Method	Organism	Contact time	
Bacteria	EN 16615	Pseudomonas aeruginosa	60 sec (c)	2 min (d)
	EN 16615	Staphylococcus aureus	60 sec (c)	2 min (d)
	EN 16615	Enterococcus hirae	60 sec (c)	2 min (d)
	EN 13727	Pseudomonas aeruginosa	30 sec (c)	
	EN 13727	Staphylococcus aureus	30 sec (c)	
	EN 13727	Enterococcus hirae	30 sec (c)	
Yeast	EN 16615	Candida albicans	60 sec (c)	2 min (d)
	EN 13624	Candida albicans	30 sec (c)	
	EN 13624	Candida auris	30 sec (c)	
Enveloped virus	EN 14476	Vaccinia Virus	30 sec (c)	60 sec (d)
	EN 14476	Human Coronavirus ATCC VR-740	30 sec (c)	
Non-enveloped virus	EN 14476	Norovirus	30 sec (c)	60 sec (d)
TB	EN 14348	Mycobacterium terrae	2 min (c)	

### Authorised distributor

#### In Australia:

For customer service, call 1300-735-292  
To email an order, [ordersanz@thermofisher.com](mailto:ordersanz@thermofisher.com)  
To order online: [thermofisher.com](http://thermofisher.com)

#### In New Zealand:

For customer service, call 0800-933-966  
To email an order, [ordersanz@thermofisher.com](mailto:ordersanz@thermofisher.com)  
To order online: [thermofisher.com](http://thermofisher.com)

**ThermoFisher**  
SCIENTIFIC

**Find out more about how our Layered Approach to Infection Prevention can help you.**

PDI Ltd, Pywell Road, Willowbrook Est. Industrial Estate, Corby - Northants, NN17 5XJ, UK

11.09.2023 V1 THERMO FISHER SCIENTIFIC

**+ PDI**  
BE THE DIFFERENCE™