



Sani-Cloth® AF is a two in one solution ideal for cleaning and disinfecting.

- + **Ready to use:** pre-dosed, alcohol free, low linting disposable cleaning and disinfection wipes, requiring no mixing or measuring.
- + **Proven efficacy:** effective against Coronavirus and Norovirus in as little as 30 seconds.
- + **Safe to use:** alcohol free, bleach free and PHMB free, causing no skin irritation or sensitisation.
- + **Improved efficiencies:** one product to clean and disinfect, providing you with consistent and reliable results.
- + **Maximum flexibility:** suitable for any healthcare environment available in flow wraps, canisters
- + **Optimal protection:** a quaternary ammonium based formulation which can be used on non-invasive medical devices; non-porous hard surfaces.
- + **Cost effective:** using the same solution for cleaning and disinfecting saves you time and money.



Product Efficacy

SANI-CLOTH® AF	TEST METHOD	ORGANISM	CONTACT TIME*	
Bacteria	EN 16615	Pseudomonas aeruginosa	60 sec (c)	2 min (d)
	EN 16615	Staphylococcus aureus	60 sec (c)	2 min (d)
	EN 16615	Enterococcus hirae	60 sec (c)	2 min (d)
	EN 13727	Pseudomonas aeruginosa	30 sec (c)	
	EN 13727	Staphylococcus aureus	30 sec (c)	
	EN 13727	Enterococcus hirae	30 sec (c)	
Yeasts	EN 16615	Candida albicans	60 sec (c)	2 min (d)
	EN 13624	Candida albicans	30 sec (c)	
	EN 13624	Candida auris	30 sec (c)	
Enveloped Virus	EN 14476	Vaccinia Virus	30 sec (c)	60 sec (d)
	EN 14476	Human Coronavirus ATCC VR-740	30 sec (c)	
Non-enveloped Virus	EN 14476	Norovirus	30 sec (c)	60 sec (d)
TB	EN 14348	Mycobacterium terrae	2 min (c)	

*Clean Conditions (c) Dirty Conditions (d)

Point of Use Accessories



Wall bracket for flow wraps

Product Code: PDIA003GRN

Made with antimicrobial properties proven to reduce the growth of bacteria by up to 99.99%.



Bracket for canisters

Bracket: PDIA001GRY

Canister brackets affix securely to the wall, holding the canister in place at its location.

Product Details

PDI Sani Cloth AF Wipes	ITEM CODE	WIPE COUNT	WIPE SIZE	QTY PER CASE
200ct Flow Wrap	PDIEAF1131E	200	267mm x 210mm	6
200ct Large Canister	PDIEAF1121E	200	200mm x 200mm	6

Other formulations available:



Authorised distributor

In Australia:

For customer service, call 1300-735-292

To email an order, ordersanz@thermofisher.com

To order online: thermofisher.com

In New Zealand:

For customer service, call 0800-933-966

To email an order, ordersanz@thermofisher.com

To order online: thermofisher.com

ThermoFisher
SCIENTIFIC

Find out more about how our Layered Approach to Infection Prevention can help you.

PDI Ltd, Pywell Road, Willowbrook Est. Industrial Estate, Corby - Northants, NN17 5XJ, UK

11.09.2023 V1 THERMO FISHER SCIENTIFIC

PDI
BE THE DIFFERENCE™