


**Sani-Cloth®**  
CLEANING AND DISINFECTANT

**Chlor**  
WIPE

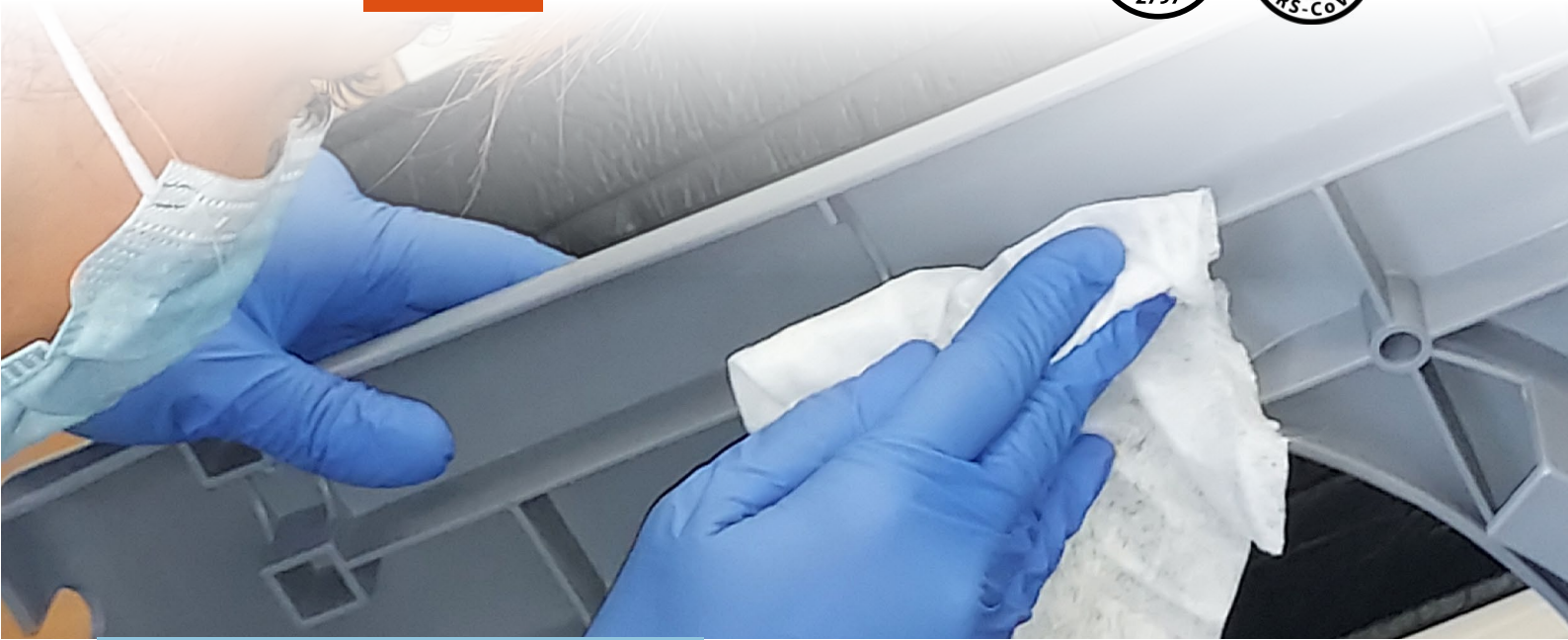

# Sani-Cloth® Chlor is a sporicidal cleaning and disinfectant solution ideal for use in high risk, contaminated environments.

- + **Ready to use:** pre-dosed, low linting disposable cleaning and disinfection wipe, requiring no mixing or measuring.
- + **Proven efficacy:** effective against all spores, including C.difficile and fully virucidal in 60 seconds.
- + **Improved efficiencies:** the product solution is impregnated with sodium hypochlorite and surfactant to clean and disinfect providing you with consistently reliable results and significant time savings delivering approximately 6000 ppm available chlorine.
- + **Optimal protection:** can be used on non-invasive medical devices; non-porous hard surfaces with supporting compatibility data.
- + **Cost effective:** using the same solution for cleaning and disinfecting saves you time and money.



**+ PDI**  
BE THE DIFFERENCE®

## Product Efficacy

SANI-CLOTH® CHLOR	TEST METHOD	ORGANISM	CONTACT TIME*	
<b>Bacteria</b>	EN 16615	Pseudomonas aeruginosa	60 sec (c)	2 min (d)
	EN 16615	Staphylococcus aureus	60 sec (c)	2 min (d)
	EN 16615	Enterococcus hirae	60 sec (c)	2 min (d)
	EN 13727	Pseudomonas aeruginosa	30 sec (c)	
	EN 13727	Staphylococcus aureus	30 sec (c)	
	EN 13727	Enterococcus hirae	30 sec (c)	
<b>Yeasts</b>	EN 16615	Candida albicans	60 sec (c)	2 min (d)
	EN 13624	Candida albicans	60 sec (c)	
	EN 13624	Candida auris	30 sec (c)	
<b>Enveloped Virus</b>	EN 14476	Vaccinia Virus	30 sec (c)	60 sec (d)
	EN 14476	Human Coronavirus		60 sec (d)
<b>Non-enveloped Virus</b>	EN 14476	Norovirus	60 sec (c)	60 sec (d)
	EN 14476	Adenovirus	60 sec (c)	60 sec (d)
	EN 14476	Poliovirus	60 sec (c)	60 sec (d)
<b>Spores</b>	ASTM E 2197-02	C.difficile	4 min	

\*Clean Conditions (c) Dirty Conditions (d)

## Point of Use Accessories



### Bracket for canisters

Bracket (colour coded): PDIA001GRY

Canister brackets affix securely to the wall, holding the canister in place at its location.

## Product Details

Sani Cloth Chlor Wipes	ITEM CODE	WIPE COUNT	WIPE SIZE	QTY PER CASE
50ct Large Canister	PDIXP00327A01	50	200mm x 195mm	6

Other formulations available:



## Authorised distributor

### In Australia:

For customer service, call 1300-735-292  
To email an order, [ordersanz@thermofisher.com](mailto:ordersanz@thermofisher.com)  
To order online: [thermofisher.com](http://thermofisher.com)

### In New Zealand:

For customer service, call 0800-933-966  
To email an order, [ordersanz@thermofisher.com](mailto:ordersanz@thermofisher.com)  
To order online: [thermofisher.com](http://thermofisher.com)

**ThermoFisher**  
SCIENTIFIC

Find out more about how our Layered Approach to Infection Prevention can help you.

PDI Ltd, Pywell Road, Willowbrook Est. Industrial Estate, Corby - Northants, NN17 5XJ, UK

11.09.2023 V1 THERMO FISHER SCIENTIFIC

**+PDI**  
BE THE DIFFERENCE™