

# Sani-Cloth® Detergent: Instructions for Use

## General surface cleaning and damp dusting wipes

Please follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines

### Preparation



### Dispense



- + Dispense the wipe, pulling away from you.
- + Close the lid when not in use.

### Clean



- + If the surface is visibly soiled, clean the surface following an S-shaped motion from top to bottom.
- + Use additional wipes as needed.
- + Use a fresh wipe for each new surface.

### Dry



- + Allow surface to air dry.

### Disposal



- + Dispose of wipe after a single use.
- + Packaging is not recyclable



### Other information:

- + Store upright in a cool dry place
- + Refer to expiry date on packaging
- + Approx. 1 wipe per 1m<sup>2</sup>

## Authorised distributor

### In Australia:

For customer service, call 1300-735-292  
To email an order, [ordersanz@thermofisher.com](mailto:ordersanz@thermofisher.com)  
To order online: [thermofisher.com](http://thermofisher.com)

### In New Zealand:

For customer service, call 0800-933-966  
To email an order, [ordersanz@thermofisher.com](mailto:ordersanz@thermofisher.com)  
To order online: [thermofisher.com](http://thermofisher.com)

**ThermoFisher**  
SCIENTIFIC

Find out more about how our Layered Approach to Infection Prevention can help you.

PDI Ltd, Pywell Road, Willowbrook Est. Industrial Estate, Corby - Northants, NN17 5XJ, UK

11.09.2023 V1 THERMO FISHER SCIENTIFIC

**+ PDI**  
BE THE DIFFERENCE™