

Super Sani-Cloth
CLEANING AND DISINFECTANT

WIPE



A two in one dual quaternary ammonium and alcohol-based formulation, ideal to use in fast-paced environments.

- + **Ready to use:** pre-dosed, premium, strong, low linting disposable cleaning and disinfection wipe, requiring no mixing or measuring, with no smear formulation.
- + **Proven efficacy:** effective against Coronavirus and Norovirus in as little as 60 seconds.
- + **Improved efficiencies:** one product to clean and disinfect, providing you with consistent and reliable results.
- + **Superior compatibility:** a quaternary ammonium, alcohol-based formulation which can be used on non-invasive medical devices; non-porous hard surfaces with supporting compatibility data available.
- + **Cost effective:** using the same solution for cleaning and disinfecting saves you time and money.



+ PDI
BE THE DIFFERENCE[®]

Product Efficacy

Super Sani-Cloth®	Test Method	Organism	Contact time	
Bacteria	EN 16615	Pseudomonas aeruginosa	60 sec (c)	60 sec (d)
	EN 16615	Staphylococcus aureus	60 sec (c)	60 sec (d)
	EN 16615	Enterococcus hirae	60 sec (c)	60 sec (d)
	EN 13727	Pseudomonas aeruginosa	30 sec (c)	60 sec (d)
	EN 13727	Staphylococcus aureus	30 sec (c)	60 sec (d)
	EN 13727	Enterococcus hirae	30 sec (c)	60 sec (d)
Yeast	EN 16615	Candida albicans	60 sec (c)	60 sec (d)
	EN 13624	Candida albicans	30 sec (c)	
	EN 13624	Candida auris	30 sec (c)	
Enveloped virus	EN 14476	Vaccinia Virus	60 sec (c)	60 sec (d)
	EN 14476	Corona Virus	60 sec (c)	60 sec (d)
Non-enveloped virus	EN 14476	Norovirus	30 sec (c)	60 sec (d)
TB	EN 14348	Mycobacterium terrae	60 sec (c)	

*Clean Conditions (c) Dirty Conditions (d)

Point of Use Accessories



Bracket for canisters

Bracket: PDIA001 GRY

Canister brackets affix securely

to the wall, holding the canister in

Product Details

SUPER SANI CLOTH	ITEM CODE	WIPE COUNT	WIPE SIZE	QTY PER CASE
100ct Large Canister	PDIESC1621E	100	200mm x 200mm	6

Other formulations available:



Authorised distributor

In Australia:

For customer service, call 1300-735-292

To email an order, ordersanz@thermofisher.com

To order online: thermofisher.com

In New Zealand:

For customer service, call 0800-933-966

To email an order, ordersanz@thermofisher.com

To order online: thermofisher.com

ThermoFisher
SCIENTIFIC

Find out more about how our Layered Approach to Infection Prevention can help you.

PDI Ltd, Pywell Road, Willowbrook Est. Industrial Estate, Corby - Northants, NN17 5XJ, UK

11.09.2023 V1 THERMO FISHER SCIENTIFIC

+PDI
BE THE DIFFERENCE™