

Thermo Scientific Richard-Allan Scientific ParaPel

Instructions for Use

For in vitro diagnostic use.

For use as a paraffin repellant on microtomes and accessories.

Thermo Scientific™ Richard-Allan Scientific™ ParaPel™ is designed to be used at full strength and should not require any additives. It is used as a paraffin repellent for countertops, microtomes and microtome accessories. This product is available in a non-aerosol pump spray bottle.

Instructions For Use

Procedure

1. Holding bottle upright, spray generously on a clean surface. Wipe dry with a clean cloth.
2. When used properly, ParaPel prevents paraffin wax from adhering to equipment and countertops.

Warnings and Precautions

See Safety Data Sheets for warnings and precautions, as well as EUH code definitions.
See container label for warnings and precautions.

Order Information

Product	Size	Qty.	REF
ParaPel	4 oz. (118 mL)	6/cs.	2300

