

# **Confidence in Results**

With Sensitivity and Quantitative Accuracy







### Be Confident in Your Test Results

Your busy clinical laboratory must quickly and cost-effectively deliver accurate test results to the healthcare providers you serve. Quantitative results for your numerous laboratory-developed tests must be fast, accurate, and reproducible. And you must be confident in these tests, even when presented with low-concentration, limited-volume samples.

The Thermo Scientific™ Endura MD™ mass spectrometer is a Class I medical device for in vitro diagnostic use designed to help you meet these demands confidently, day after day. It combines excellent quantitative performance, sensitivity, and robustness with built-in ease-of-use to help you deliver more value and worry-free results.


Combine the Endura MD mass spectrometer with the Thermo Scientific Prelude MD HPLC and Thermo Scientific ClinQuan MD software, and your laboratory can obtain the quantitative accuracy of LC-MS, easily and confidently.

# Access the Quantitative Accuracy of LC-MS Easily and Confidently

### Quantitate Low-level Samples Accurately

To ensure accurate quantitation of limited-volume, low-concentration samples, the Endura MD mass spectrometer delivers high sensitivity even when analyzing complex matrices. Designed to reduce noise and increase sensitivity, the ion optics stay clean longer for more consistent results. This triple-quadrupole mass spectrometer with ultrafast selected reaction monitoring (SRM) increases the speed of analysis without sacrificing quantitative accuracy. Five orders of linear dynamic range further boost confidence in results.


Less than 3% RSDs obtained for five injections of three different QC samples show excellent precision across a range of concentrations of tacrolimus, sirolimus and cyclosporine A.

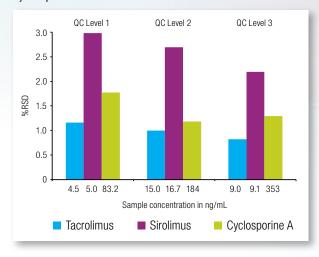

## Run Samples Reliably Day After Day

Robust and reliable, the Endura MD mass spectrometer is designed to run numerous samples day after day. To provide consistent performance, the ion source exhaust removes more solvent vapor, the neutral blocker keeps the ion optics cleaner, and the advanced electronics maintain mass accuracy and stability longer.


Analysis of a clomipramine standard spiked in human plasma at a LOQ of 1 ng/mL showing quantifying ion, internal standard ion, and calibration curve.

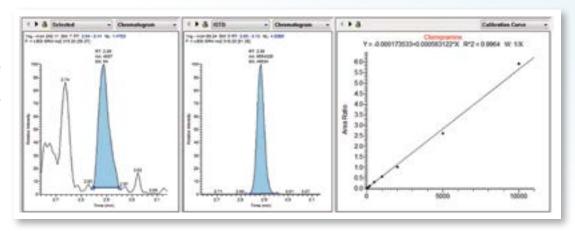



Over 1200 injections of alprazolam standard spiked in crashed synthetic serum demonstrate excellent stability.

The Endura MD mass spectrometer provides sensitive analysis of ethyl sulfate and ethyl glucuronide standards spiked in urine.

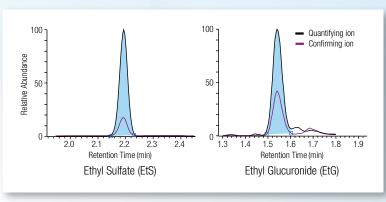

## Use the Instrument Quickly and Easily

Operators can be quickly trained to run the instrument and to obtain accurate results. Designed for user-friendly set up and operation, the Thermo Scientific™ Easy-Max NG™ ion source makes most gas and electrical connections automatically when installed.

To increase the range of target compounds that can be analyzed, the instrument accommodates both heated electrospray ionization (HESI) and atmospheric pressure chemical ionization (APCI) probes, which can be operated in either positive or negative ion modes. The probes can be swapped quickly without tools.



#### Prelude MD HPLC

Built to make LC-MS easier, the Prelude MD HPLC automates sample cleanup using Thermo Scientific™ TurboFlow™ technology. TurboFlow technology greatly reduces complicated manual sample cleanup steps and variability, so you can be more confident in results. To maximize sample throughput, the HPLC features two parallel channels that can run identical or different methods at the same time.

#### Endura MD Mass Spectrometer

The Endura MD mass spectrometer combines usability, excellent quantitative performance, sensitivity, and robustness to deliver high-confidence LC-MS results day after day. The instrument features accurate, reliable quantitation and stable performance for the analysis of even small-volume and low-concentration samples.





For in vitro diagnostic use. Not available in all countries. Please visit www.thermoscientific.com/enduraMD For complete product line information visit www.thermoscientific.com

Thermo Finnigan LLC, a part of Thermo Fisher Scientific Inc. 355 River Oaks Parkway, San Jose, CA 95134, United States Tel. (408) 965-6000 Printed in the United States.

©2014 Thermo Fisher Scientific Inc. All rights reserved. ISO is a trademark of the International Standards Organization. All other trademarks are the property of Thermo Fisher Scientific and its subsidiaries. This information is presented as an example of the capabilities of Thermo Fisher Scientific products. It is not intended to encourage use of these products in any manners that might infringe the intellectual property rights of others. Specifications, terms and pricing are subject to change. Not all products are available in all countries. Please consult your local sales representative for details.

Africa +43 1 333 50 34 0 **Australia** +61 3 9757 4300 Austria +43 810 282 206 Belgium +32 53 73 42 41 Canada +1 800 530 8447

China 800 810 5118 (free call domestic) 400 650 5118

**Denmark** +45 70 23 62 60 **Europe-Other** +43 1 333 50 34 0 Finland +358 9 3291 0200 France +33 1 60 92 48 00 Germany +49 6103 408 1014 India +91 22 6742 9494 Italy +39 02 950 591

**Japan** +81 45 453 9100 Korea +82 2 3420 8600 **Latin America** +1 561 688 8700 Middle East +43 1 333 50 34 0 Netherlands +31 76 579 55 55  $\textbf{New Zealand} \hspace{0.2cm} +64 \hspace{0.1cm} 9 \hspace{0.1cm} 980 \hspace{0.1cm} 6700$ Norway +46 8 556 468 00

For in vitro diagnostic use. Not available in all countries.



Thermu i i.c.
San Jose, CA USA is
ISO 13485 Certified. Thermo Fisher Scientific.

Russia/CIS +43 1 333 50 34 0 **Singapore** +65 6289 1190 **Spain** +34 914 845 965 **Sweden** +46 8 556 468 00 **Switzerland** +41 61 716 77 00 **UK** +44 1442 233555 USA +1 800 532 4752



A Thermo Fisher Scientific Brand