applied biosystems



Guidelines for taking diagnostic samples from pigs

Tissue

A series of best practices leaflets developed in conjunction with Dr. Heiko Nathues, Royal Veterinary College, UK



Diagnostic use

Detection of pathogens by microscopy—Some protozoa and several helminth parasites or their eggs (e.g., Ascaris suis, Balantidum coli, Strongyloides ransomi, Isospora suis, Oesophagostomum dentatum, Trichuris suis, and others) as well as some viruses (e.g., rotaviruses, coronaviruses, and others) can be identified as suspicious using different microscopic techniques with tissue samples of the small and large intestine, respectively. Submission of other tissue samples to microscopy for the identification of pathogens is possible, but seldom performed.

Detection of pathogens by cultural testing—Tissue samples can be tested by culture for the presence of a range of bacterial respiratory, enteric, reproductive and/or systemic pathogens. All bacteria that can be isolated from other samples, e.g., BALF, feces, nasal swabs, synovia, and others can also be isolated from the corresponding tissue.

Detection of pathogen RNA/DNA by PCR-based tests-

The presence of pathogens can be confirmed in various tissues. Detection of bacteria or viruses by highly sensitive PCR techniques does not establish clinical relevance for organisms that may be considered normal flora in certain animal populations.

Animal selection

Deciding which animals to take samples from depends on the desired outcome:

- Cause of death—Take tissue samples from the pigs that died spontaneously. Be aware of autolysis, which precludes those pigs from examination that have died more than 24 hours ago.
- Cause of disease—Take tissue samples from pigs that show the typical symptoms of the particular disease, preferably in the acute phase. Do not take samples from pigs showing stunted growth as the only sign, or from pigs showing additional signs of another disease that is of minor interest.

Sample size

In either case (spontaneous deaths or clinical disease), at group level, a minimum of 3 pigs is recommended for postmortem examination and subsequent tissue sampling. If symptoms are not consistent in the affected group, select at least 3 pigs per "typical symptom" (e.g., in a group with pigs showing either coughing or central nervous symptoms, select 3 pigs showing coughing as the main symptom and 3 pigs showing central nervous signs as the main symptoms).

Sample sizes may vary based on in-herd prevalence level of a disease, the tested disease itself, confidence level of the outcome, the requested test method and the purpose of the sampling.

Preparation

- When selecting live animals, separate these from the others before euthanizing them. Use electro-stunning whenever possible. If this is not used, intravenous injection of anesthetizing drugs (e.g., ketamine plus azaperone) followed by an overdose of barbiturate, is most appropriate. Do not kill animals by head shot if further examination of brain tissue is the aim.
- Do not take tissue samples in areas stocked with live animals.
- Perform the necropsy in a clean area with concrete floor and enough light. This place should be cleaned and disinfected immediately after the necropsy and should not be accessible to other pigs.

Sampling technique

- 1. Open the different body cavities separately; start with the abdominal cavity, followed by the thoracic cavity, and end with the skull cavity.
- 2. Use a cleaned and disinfected knife (saw for the head). Clean and disinfect the knife between successive pigs.
- 3. Take samples at the visible border between affected and unaffected tissue.
- 4. Use a sterile scalpel blade for sectioning the tissue. The samples should be approx. 2 cm along each edge. If different organs are affected, use at least one new blade per organ system. When sampling gut tissue, ligate both ends of the anticipated sample before cutting.
- Use sterile collection tubes for each tissue sample (10–50 mL). Tubes must not contain any salts, etc.







- 6. Carefully withdraw scalpel blades after use. Do not put them into the sample tubes.
- 7. Label the tube immediately with the animal ID (ear tag number) and the type of tissue using a waterproof marker. Write numbers and letters clearly according to good clinical practice.

Storage

Samples should be stored in a refrigerator until shipment to the laboratory, which should be within 24–36 hours. If this is not possible and only PCR is required, freeze the sample at –20 to –80°C. Keep in mind that no further cultural examination is possible after freezing a sample.

Shipment

Material from diseased animals is usually classified as "Biological substance, category B" according to UN regulations (UN 3373). It must be shipped in compliance with national regulations and, at least for international shipment, in compliance with "Packing Instruction 650" specified by the International Air Transport Association (IATA). National regulations and IATA instructions may change over time. If you have doubt about the actual regulations, please ask your courier or the lab.

The sample should be accompanied by a case history and examination form, including:

- Name of veterinarian
- Name of farmer/herd owner
- Invoicing information
- Species/breed and age of sampled animals
- Date samples were taken
- Number of samples
- Type of samples
- Identification/labeling of samples (correlation between numbers on the samples and ear tags on pigs)
- Specified test that should be performed, such as "quantitative real-time PCR for *Lawsonia intracellularis*" rather than just "L.i. detection"; or "electron microscopy for rota and coronaviruses" rather than just "Rota/Corona"
- Results from any previous tests that do not need to be repeated

Good background information can help the laboratory conduct the most appropriate tests and provide advice in context.

applied biosystems

This brochure is for informational purposes only. It is the responsibility of the customer to perform his or her own analysis and appropriate internal verification studies to ensure that the customer's processes and workflows, including products and services obtained from Thermo Fisher Scientific, satisfy or will satisfy the customer's requirements and are fit for the customer's animal health applications.

For more information, contact your farm animal diagnostic testing laboratory, or go to **thermofisher.com/animalhealth**



For Veterinary Use Only. For In Vitro Use Only. Regulatory requirements vary by country, products may not be available in your geographic area. © 2015–2019 Thermo Fisher Scientific Inc. All rights reserved. All trademarks are the property of Thermo Fisher Scientific and its subsidiaries unless otherwise specified. COL010808 1219