RapID™ Nitrate A Reagent and RapID Nitrate B Reagent

REF R8309003, RapID Nitrate A Reagent 15 ml/Bottle

REF R8309004, RapID Nitrate B Reagent 15 ml/Bottle

1. INTENDED USE

RapID Nitrate A and RapID Nitrate B are reagents to be used with the RapID manual identification systems.

The devices are for professional use only.

2. SUMMARY AND EXPLANATION

Nitrate is a source of nitrogen for many bacteria. The ability to reduce nitrate to nitrite is characteristic of the family Enterobacteriaceae.¹ Non-fermentative Gram-negative bacilli and other miscellaneous Gram-negative and Gram-positive bacteria vary in their ability to reduce nitrate. Nitrate reduction results in several end-products depending on the bacterial species being tested.²

3. PRINCIPLE

The nitrate reduction test is a two-step process. RapID Nitrate A Reagent is added to acidify the medium followed by RapID Nitrate B Reagent which reacts with nitrite to produce a red colour. Some organisms further reduce nitrites to form nitrogen gas or ammonia. If no colour develops following the addition of both reagents, the organism is either not capable of reducing nitrate or it has further reduced nitrite to free nitrogen gas.³

4. REAGENTS

RapID Nitrate A Reagent 15ml/Bottle

Sulfanilic Acid (CAS 121-57-3) 8.0 g

Glacial Acetic Acid (CAS 64-19-7) 280.0 ml

Demineralized Water (CAS 7732-18-5) 720.0 ml

RapID Nitrate B Reagent 15ml/Bottle

N,N-dimethyl-1-naphthylamine (CAS 86-56-6) 6.0 g

Glacial Acetic Acid (CAS 64-19-7) 280.0 ml

Demineralized Water (CAS 7732-18-5) 720.0 ml

5. PRECAUTIONS

DANGER! Causes severe digestive and respiratory tract burns. Causes severe eye and skin burns.

This product is for *in vitro* diagnostic use and should be used by properly trained individuals. Precautions should be taken against the dangers of microbiological hazards by properly sterilizing specimens, containers and media after use. Directions should be read and followed carefully. Refer to Material Safety Data Sheet (SDS) on company website for additional information.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the

competent authority of the Member State in which the user and/or the patient is established.

In the event of malfunction do not use device.

Nitrate A

Composition / information on ingredients

Acetic acid 64-19-7

Sulfanilic acid 121-57-3

Nitrate B

Composition / information on ingredients

Acetic acid 64-19-7

N,N-dimethyl-1-naphthylamine - 86-56-6

DANGER

US ONLY

US ONLY

US & EU

| | |
|-----------------|---|
| H226 | Flammable liquid and vapor |
| H314 | Causes severe skin burns and eye damage |
| H335 | May cause respiratory irritation |
| P260 | Do not breathe dust/fume/gas/mist/vapors/spray |
| P264 | Wash face, hands and any exposed skin thoroughly after handling |
| P280 | Wear protective gloves/protective clothing/eye protection/face protection |
| P271 | Use only outdoors or in a well-ventilated area |
| P210 | Keep away from heat/sparks/open flames/hot surfaces. - No smoking |
| P233 | Keep container tightly closed |
| P240 | Ground/bond container and receiving equipment |
| P241 | Use explosion-proof electrical/ventilating/lighting/equipment |
| P242 | Use only non-sparking tools |
| P243 | Take precautionary measures against static discharge |
| P235 | Keep cool |
| P310 | Immediately call a POISON CENTER or doctor/physician |
| P304+P340 | IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing |
| P303+P361 +P353 | IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower |
| P363 | Wash contaminated clothing before reuse |
| P305+P351 +P338 | IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing |
| P301+P330 +P331 | IF SWALLOWED: Rinse mouth. DO NOT induce vomiting |
| P370+P378 | In case of fire: Use CO ₂ , dry chemical, or foam for extinction |
| P405 | Store locked up |
| P403+P233 | Store in a well-ventilated place. Keep container tightly closed |
| P501 | Dispose of contents/container to an approved waste disposal plant |

Emergency Telephone Number

INFOTRAC - 24 Hour Number: 1-800-535-5053

Outside of the United States, call 24 Hour Number: 001-352-323-3500 (Call Collect)

6. STORAGE

This product is ready for use and no further preparation is necessary. Store product in its original container at 2-8°C until used.

7. PRODUCT DETERIORATION

This product should not be used if (1) the colour has changed, (2) the expiration date has passed, or (3) there are other signs of deterioration.

8. SPECIMEN COLLECTION, STORAGE, AND TRANSPORT

Specimens should be collected and handled following recommended guidelines.³

9. MATERIALS REQUIRED BUT NOT SUPPLIED

(1) Loop sterilization device, (2) Inoculating loop, swabs, collection containers, (3) Incubators, alternative environmental systems, (4) Supplemental media, (5) Quality control organisms.

10. QUALITY CONTROL

All lot numbers of RapID Nitrate A Reagent and RapID Nitrate B Reagent have been tested and found to be acceptable. Quality control organisms should be tested according to the guidelines in the RapID Identification System in use or according to established laboratory quality control procedures. If aberrant quality control results are noted, patient results should not be reported.

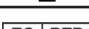
11. BIBLIOGRAPHY

1. Ewing, W.H. 1986. Edwards & Ewing's Identification of Enterobacteriaceae. 4th ed. Elsevier, New York, NY.
2. MacFaddin, J.F. 2000. Biochemical Tests for the Identification of Medical Bacteria. 3rd ed. Lippincott Williams & Wilkins, Philadelphia, PA.
3. Carroll, K C, M A Pfaller et al., 2019. Manual of Clinical Microbiology: Twelfth Edition (ASM Books)

12. PACKAGING

REF R8309003, RapID Nitrate A Reagent 15 ml/Bottle
REF R8309004, RapID Nitrate B Reagent 15 ml/Bottle

13. SYMBOL LEGEND

| | |
|---|---|
|  | Catalogue Number |
|  | In Vitro Diagnostic Medical Device |
|  | Consult Instructions for Use (IFU) |
|  | Temperature Limitations (Storage temp.) |
|  | Do not use if package is damaged |
|  | Not for near patient testing |
|  | Batch Code (Lot Number) |
|  | Use By (Expiration Date) |
|  | Authorized representative in the European Community |
|  | UK Conformity Assessed |
|  | European Conformity Assessment |
|  | Manufacturer |

For technical information contact your local distributor.

©2022 Thermo Fisher Scientific Inc. All rights reserved.

All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

CAS (Chemical Abstracts Service Registry No.)

IFU8309003, Revised March 2022

Printed in UK

12076 Santa Fe Trail Drive, Lenexa, KS 66215, USA

www.thermofisher.com/microbiology

Tel: (800) 255-6730 • International: (913) 888-0939

www.oxid.com/IFU

Europe +800 135 79 135 • US 1 855 2360 190

CA 1 855 805 8539 • ROW +31 20 794 7071

RapID™ Nitrate A Reagent and RapID Nitrate B Reagent

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©2022 Thermo Fisher Scientific Inc. Reservados todos los derechos.
Todas las demás marcas comerciales son propiedad de Thermo Fisher Scientific Inc. y sus filiales.