

A trial was conducted with 562 prospectively collected specimens tested by O&P. The specimens were collected from a large metropolitan hospital reference laboratory (360 unpreserved specimens) and from a public health laboratory (202 specimens preserved in formalin). There was one O&P positive/EIA negative result and 10 specimens were O&P negative/EIA positive. One of these was GSA 65 positive by specific inhibition. The performance of the ProSpecT Giardia Microplate Assay in this trial is given in the following chart:

| | | O&P/Specific Inhibition | | |
|----------|---|-------------------------|-----|-----|
| | | + | - | |
| ProSpecT | + | 42 | 9 | |
| Giardia | - | 1 | 510 | |
| | | 43 | 519 | 562 |

Sensitivity 42/43 = 98% (88 - 100%)

Specificity 510/519 = 98% (97 - 99%)

Numbers in parentheses are 95% confidence intervals.

ANALYTICAL SENSITIVITY

The ProSpecT Giardia Microplate Assay detects approximately 3.9 nanograms/ml of GSA 65.

REPRODUCIBILITY

The inter-assay or run-to-run coefficient of variation (CV) was evaluated with 5 positive and 5 negative samples assayed at least ten times in three separate runs. For the dilution in tube procedure the mean CV of the negative samples was 3.02% (range 0.93% to 3.90%) and the mean CV for the positive samples was 4.65% (range 1.74% to 10.72%). For the dilution in wells procedure the mean CV of the negative samples was 7.66% (range 4.22% to 16.19%) and the mean CV for the positive samples was 7.76% (range 4.12% to 11.26%).

The intra-assay or within-run CV was evaluated with 5 positive and 5 negative samples assayed at least ten times in a single run. For the dilution in tube procedure the mean CV of the negative samples was 4.47% (range 3.30% to 5.39%) and the mean CV for the positive samples was 5.61% (range 2.53% to 10.12%). For the dilution in wells procedure the mean CV of the negative samples was 4.51% (range 2.73% to 5.36%) and the mean CV for the positive samples was 9.58% (range 4.78% to 13.8%).

CROSS-REACTIVITY

The ProSpecT Giardia Microplate Assay has been tested with stool specimens found to be positive for a number of faecal organisms. No cross-reactivity was observed with any of the infectious agents listed below.

| | |
|------------------------------------|--------------------------------------|
| <i>Ascaris lumbricoides</i> (5) | <i>Entamoeba histolytica</i> (5) |
| <i>Blastocystis hominis</i> (6) | <i>Hymenolepis nana</i> (2) |
| <i>Cryptosporidium parvum</i> (10) | <i>Iodamoeba butschlii</i> (9) |
| <i>Dientamoeba fragilis</i> (10) | <i>Isospora belli</i> (5) |
| <i>Endolimax nana</i> (6) | Rotavirus (11) |
| <i>Entamoeba coli</i> (13) | <i>Strongyloides stercoralis</i> (1) |
| <i>Entamoeba hartmanni</i> (5) | <i>Trichuris trichiuria</i> (2) |

Numbers in parentheses indicate the numbers of specimens tested.

14 BIBLIOGRAPHY

- Addiss, D.G., 1991.**
Evaluation of a commercially available Enzyme-Linked Immunoabsorbent Assay for *Giardia lamblia* antigen in stool. J. Clin. Microbiol. 29(6):1137-1142.
- Black, R.E. et al., 1977.**
Giardiasis in day care centres: evidence of person to person transmission. Pediatrics 60:486-491.
- Centre for Disease Control, Atlanta, GA, 1978.**
Intestinal parasite surveillance-United States 1976. Morbid. Mortal. Weekly Rep. 27(20):167-168.
- Craun, G.F., 1979.**
Waterborne Giardiasis in the United States: A Review. Am. J. Publ. Health 69(8):817-819.
- Craun, G.F., 1986.**
Waterborne Giardiasis in the United States 1965-84. Lancet 1:513-514.
- Kean, B.H. et al., 1979.**
Epidemic of amoebiasis and giardiasis in a biased population. Brit. J. Ven. Dis. 55:375-378.
- Meyers, J.D. et al., 1977.**
Giardia lamblia infection in homosexual men. Brit. J. Ven. Dis. 53:54-55.
- Rendtorff, R.C., 1979.**
The experimental transmission of *Giardia lamblia* among volunteer subjects in waterborne transmission of Giardiasis (Jakubowski, W., and Hoff, J.C., eds.), pp. 64-81. EPA, U.S., Cincinnati, OH.
- Rosoff, J.D., and H.H. Stibbs, 1986.**
Isolation and Identification of a *Giardia lamblia*-Specific Stool Antigen (GSA 65) useful in the coprodiagnosis of Giardiasis. J. Clin. Microbiol. 23(5):905-910.
- Rosoff, J.D., and H.H. Stibbs, 1986.**
Physical and chemical characterization of a *Giardia lamblia*-Specific Antigen Useful in the coprodiagnosis of Giardiasis. J. Clin. Microbiol. 24(6):1079-1083.
- Rosoff, J.D. et al., 1989.**
Stool Diagnosis of Giardiasis using a commercially available Enzyme Immunoassay to Detect *Giardia*-Specific Antigen 65 (GSA 65). J. Clin. Microbiol. 27(9):1997-2002.
- Schieven, B.C. and Z. Hussain, 1990.**
Evaluation of an enzyme immunoassay test kit for diagnosing infections with *Giardia lamblia*. Serodiag and Immunother 4:109-113.
- Sonnad, S., L. Bahrami, P. O'Hanley, 1991.**
Compatibility Assessment of Three Common Transport Media Systems with the ProSpecT™ / *Giardia* Immunoassay. Presented at the 1991 American Society for Microbiology Meeting, Session 20.
- Tijssen, P.**
Practice and Theory of Enzyme Immunoassays. Laboratory Techniques in Biochemistry and Molecular Biology, R.H. Burdon and P.H. van Knippenberg, eds., Elsevier, N.Y., 1985, pp. 14-16.
- Visvesvara, G.S., 1982.**
Giardiasis in Children. J. Pediatr. Gastroentrol Nutr. 1(4):463-465.
- Wolfe, M.S., 1979.**
Managing the patient with giardiasis: clinical, diagnostic and therapeutic aspects. In: Waterborne Transmission of Giardiasis. (Jakubowski, W., and Hoff, J.C. eds.) pp. 39-52. EPA, U.S., Cincinnati, OH.

ProSpecT™ is a registered trademark



Oxoid Ltd Wade Road Basingstoke
Hants, RG24 8PW UK

For technical assistance please contact your local distributor.

IFU X7588A, Revised March 2012

Printed in the UK