

from the trial data, as follows:

	PathoDxtra Streptococcal Grouping Reagents	Predicate device
Mean sensitivity (%)	89.1	85.2
Mean specificity (%)	97.8	97.5

13. BIBLIOGRAPHY

- 1 Murray, P.R., E.J. Baron, J.H. Jorgensen, M.A. Pfaller, and R.H. Tenenbaum, 2003. Manual of Clinical Microbiology. 8th ed., Vol 1. ASM, Washington, D.C.
- 2 Evins, G.M., et al., 1983. The development by the Centers for Disease Control of a specification for streptococcal serogrouping kits and its application to Streptex and to the Phadebact Streptococcus Test. J. Biol. Standard. 11:333-339.
- 3 Facklam, R.R., et al., 1979. Evaluation of commercial latex agglutination reagents for grouping streptococci. J. Clin. Microbiol. 10:641-646.
- 4 Slifkin, M., and G.R. Pouchet-Melvin, 1980. Evaluation of three commercially available test products for serogrouping beta-hemolytic streptococci. J. Clin. Microbiol. 11:249-255.
- 5 Harvey, C.L., and McIlmurray, M.B., 1984. Streptococci with dual antigen specificity for Lancefield groups D and G. Eur. J. Clin. Microbiol., 3, 526.
- 6 Jablon, J.M., et al., 1965. β -Hemolytic Streptococci with Group A and Type II Carbohydrate Antigens. J. Bacteriol. 89:529-534.
- 7 Ottens, H., and K.C. Winkler, 1962. Indifferent and Haemolytic Streptococci Possessing Group-Antigen F. J. Gen. Microbiol. 28:181-191
- 8 Jennings, H.L., et al., 1980. Structure of the complex polysaccharide C-substance from Streptococcus pneumoniae type 1. Biochem. 19:4712-4719.
- 9 Krause, R.M., and M. McCarty, 1962. Studies on the Chemical Structure of the Streptococcal Cell Wall: II The Composition of Group C Cell Walls and Chemical Basis for Serologic Specificity of the Carbohydrate Moiety. J. Exp. Med. 115:49-62
- 10 Poxton, I.R., et al., 1978. The structure of C-polysaccharide from the walls of Streptococcus pneumoniae. Biochem. J. 175:1033-1042.
- 11 Lee, P., and B.L. Wetherall, 1987. Cross-reaction between Streptococcus pneumoniae and group C streptococcal latex reagent. J. Clin. Microbiol. 25:152-153.
- 12 Hopfer, R.L., et al., 1985. Enzyme release of antigen from Streptococcus faecalis and Listeria monocytogenes cross-reactive with Lancefield group G typing reagents. J. Clin. Microbiol. 22:677-679.
- 13 Shlaes, D.M., et al., 1984. Comparison of latex agglutination and immunofluorescence for direct Lancefield grouping of streptococci from blood cultures. J. Clin. Microbiol. 20:195-198.
- 14 Wellstood, S., 1982. Evaluation of Phadebact and Streptex Kits for rapid grouping of streptococci directly from blood cultures. J. Clin. Microbiol. 15:226-230.
- 15 Wetkowski, M.A., et al., 1982. Direct Testing of Blood Cultures for Detection of Streptococcal Antigens J. Clin. Microbiol. 16:86-91.
- 16 Facklam, R.R., 1977. Physiological differentiation of viridans streptococci. J. Clin. Microbiol. 5:184-201.
- 17 Chorprenning, F.W., Cooper, H.R., et al., 1975. Cross reactions of Streptococcus mutans Due to Cell Wall Teichoic Acid. Infect. Immun., 12, 586.
- 18 Ellio, S.D., and Taj, J.Y., 1978. The type-specific polysaccharides of Streptococcus suis. J. Exp. Med., 148, 1699
- 19 Nowlan, S.S., and Deibel, R.H., 1967. Group Q Streptococci. 1 Ecology, Serology, Physiology and Relationship to Established Enterococci. J Bact., 94, 291.
- 20 Facklam, R.R., 1984. The major differences in the American and British Streptococcus taxonomy schemes with special reference to Streptococcus milleri. Eur. J. Clin. Microbiol. 2:91-93.
- 21 Lawrence, J., et al., 1985. Incidence and characterization of beta-hemolytic Streptococcus milleri and differentiation from S. pyogenes (group A), S. equisimilis (group C), and large-colony group G streptococci. J. Clin. Microbiol. 22:772-777

All trademarks are the property of Thermo Fisher Scientific or its subsidiaries.

