



Key Code TSMX7777

www.oxid.com/ifu

Europe + 800 135 79 135

US 1 855 236 0910

CA 1 855 805 8539

ROW +31 20 794 7071

remel PathoDx® Strep D

EN

Grouping and Control Latexes

INTENDED USE

Remel PathoDx® Strep D supplementary reagents are intended for use in the PathoDx® Strep D Kit, which contains all components necessary for the various systems. These reagents are provided as replacement items. **For complete information and correct use of this product, please refer to the Instructions For Use which accompanies the PathoDx® Strep D Kit (R62036) and the Strep D/PYR Kit (R62033).**

DESCRIPTION, PREPARATION FOR USE AND RECOMMENDED STORAGE CONDITIONS

See also **Warnings and Precautions**



Store at 2 to 8°C.

LATEX

Strep D Latex (Grey cap)

Dropper bottle contains 3.0 ml of rabbit anti-Group D-specific IgG-sensitized latex suspension with 0.098% sodium azide and 0.05% ProClin 300® as preservatives. Store at 2 to 8°C. Use on or before the expiration date marked on the label. Just before use, resuspend the beads by gently vortexing the bottle or by inversion.



Negative Control Latex (Rose cap)

Dropper bottle contains 3.0 ml of non-immune rabbit IgG-sensitized latex suspension with 0.098% sodium azide and 0.05% ProClin 300® as preservatives. Store at 2 to 8°C. Use on or before the expiration date marked on the label. Just before use, resuspend the beads by gently vortexing the bottle or by inversion.

WARNINGS AND PRECAUTIONS

IVD

The reagents are for *in vitro* diagnostic use only.

For professional use only.

Please refer to the Material Safety Data Sheet and the product labeling for information on potentially hazardous components.

HEALTH AND SAFETY INFORMATION

1. Sodium azide, at concentrations of less than 0.1%, has been added to the latex reagents as an antimicrobial agent. To prevent buildup of explosive metal azides in lead and copper plumbing, reagents should be discarded into sewerage only if diluted and flushed with large volumes of water.
2. Waste Material: Sterilize all waste material and clinical specimens before discarding according to standard laboratory procedures and local regulations.
3. The reagents contain 0.05% ProClin 300®. If any of the reagents come into contact with the skin or eyes, wash the area extensively with water.

| | |
|----|---|
| Xi | Irritant |
| | R43 May cause sensitization by skin contact |
| | S24/25 Avoid contact with skin and eyes |
| | S37 Wear suitable gloves |

PACKAGING

| | |
|------------|-----------------------------------|
| REF | R62038.....Strep D Grouping Latex |
| | R62048.....Negative Control Latex |

Symbol Legend

| | |
|---------------|--|
| REF | Catalog Number |
| IVD | In Vitro Diagnostic Medical Device |
| LAB | For Laboratory Use |
| | Consult Instructions for Use (IFU) |
| | Temperature Limitation (Storage Temp.) |
| LOT | Batch Code (Lot Number) |
| | Use By (Expiration Date) |
| EC REP | European Authorized Representative |



Manufacturer



PathoDx® is a trademark of Remel Inc.

ProClin 300® is a trademark of Rohm and Haas Corp.

IFU X7777, Revised November 2011

Printed in the UK



Remel Inc. 12076 Santa Fe Drive, Lenexa, KS 66215
USA



Remel Europe Ltd., Clipper Boulevard West,
Crossways, Dartford, Kent, DA2 6PT, UK

For technical assistance please contact your local distributor.