









12. BIBLIOGRAPHY

1. Beem, M.O. and E.M. Saxon. 1977. N. Engl. J. Med. 296:306-310.
2. Bowie, W.R., W. San-Pin, E.R. Alexander, J. Floyd, P.S. Forsyth, H.M. Pollock, J.L. Lin, T. M. Buchanan, and K.K. Holmes. 1977. J. Clin. Invest. 59:735-742.
3. Brunham, R.C., I.W. Maclean, B. Binns, and R.W. Peeling. 1985. J. Infect. Dis. 152:1275-1282.
4. Curran, J.W. 1980. Amer. J. Obstet. Gynecol. 138:848-851.
5. Hammerschlag, M.R., M. Anderda, D.Z. Semine, D. McComb, and W.M. McCormack. 1979. Pediatrics. 64:142-148.
6. Kuo, C., S.P. Wand, B.B. Wentworth, and J.T. Grayston. 1972. J. Infect. Dis. 125:665-668.
7. Krieg, N.R. and J.G. Holt. 1984. Bergey's Manual of Systematic Bacteriology. Vol. 1. Williams and Wilkins, Baltimore, MD.
8. Paavonen J., et al. 1978. J. Clin. Invest. 59:735-742.
9. Ripa, K.T. and P. Mardh. 1977. J. Clin. Microbiol. 6:328-331.
10. Sabet, S.F., J. Simmons, and H.D. Caldwell. 1984. J. Clin. Microbiol. 20: 217-222.
11. Sawicki, L. 1980. Lab. Man. 18(11):42-47.
12. Schachter, J. 1978. N. Engl. J. Med. 298:428-435, 490-495, 540-549.
13. Schachter, J. and H.D. Caldwell. 1980. Ann. Rev. Microbiol. 34:285-309.
14. Sierra, M.F., et al. Lab. Med. 1988. 19:311-314.
15. Thompson, S.E. and A.E. Washington. 1983. Epidemiologic Rev. 5: 96-123.
16. Westrom, L. 1980. Amer. J. Obstet. Gynecol. 138:880-892.

13. PACKAGING

REF R62220.....100 Tests/Kit

Symbol Legend

	Catalog Number
	In Vitro Diagnostic Medical Device
	For Laboratory Use
	Consult Instructions for Use (IFU)
	Temperature Limitation (Storage Temp.)
	Batch Code (Lot Number)
	Use By (Expiration Date)
	Manufacturer

PathoDx is a registered trademark of Remel Inc.

ATCC® is a registered trademark of American Type Culture Collection.

IFU X7785 Revised November 2012

Printed in the UK

 Remel Europe Ltd.
Clipper Boulevard West, Crossways
Dartford, Kent, DA2 6PT UK

For technical assistance, please contact your local distributor.